

Product Information Guide



Live Life Community Based/Mobile Pendant Alarm

(Supplier: Live Life)

Monitoring	Family monitored
Ongoing Costs <i>(correct at date below, subject to change as per suppliers' advice)</i>	\$75/year Telstra SIM (first year covered) Optus available by request for same cost of \$75/year
Colour	Black or white
Case material	Plastic
Accessories	Charging station, lanyard, wrist band, belt clip (all included)
Range/Reception	Community use with mobile coverage (2-3 bars reception required)
Water resistance	IPX7 - Shower/splash proof
Dimensions (H x W x D)	61mm x 42mm x 16mm
Weight	42g
Battery life	Up to 4-5 days (daily charging recommended) Charging station has 8-hour battery backup
Charging time	45 minutes
Fall detection	Yes
Live tracking/GPS	Yes
Two-way call	Yes
Geofence	Yes
No Movement Detector Alarm	Yes
Voice Prompts, Languages	Yes, English (default), Mandarin, Cantonese, Greek, Italian, Vietnamese, Korean

Pacemaker information	Consult Cardiologist. May be suitable with longer lanyard; or wear on wrist strap/belt clip (<i>accessories available separately on geat2GO</i>)
Emergency contacts	Can have up to 6 contacts for family monitoring
Set up needed?	Connect charger to power and lanyard to pendant
SOS procedure	<ol style="list-style-type: none"> 1. Press button until device vibrates. 2. Voice prompt ""Alarm has been activated. Press the SOS heart button to cancel"" (approx 15 sec) 3. SMS sent to all emergency contacts simultaneously with GPS location via Google Maps and some information about the device inc battery life. Setting to only SMS contact 1 before calling can be requested. 4. Will sequentially call all emergency contacts and will loop once. Max outgoing call length 3 minutes. 5. User is required to press SOS button to stop the calling process.
How to cancel alarm	Single press of the SOS button
Additional settings/options	
Comments/comparison to other alarms	Same design and features to Suresafe (smallest and lightest pendant available). Alternative colour options to Live Life. Offers O ring to reduce accidental presses and increase tactile feedback.

Information correct as at 30 October 2023